



















































SITUATION

"Open the catheter packaging a bit."









SITUATION

"Turn the knob to the water symbol and pump 2-3 times so that water fills the catheter pack.

Count slowly to 10 three times before you do anything else.

That will make the catheter soft and smooth."





Coloplast

SITUATION

"Turn the knob to the balloon symbol. Now you are ready to pump the balloon on the catheter."







**SITUATION** 

"Put the catheter slowly and gently into your bottom.
When the catheter is all the way in, pump the balloon while you hold the catheter in place. The air in the balloon makes the catheter stay in place so you don't need to hold it anymore."





SITUATION

"Turn the knob to the water symbol and very slowly pump water into your bottom. Ask your parents how much water you need to pump."







SITUATION

"Turn the knob to the air symbol to let out air from the balloon. The catheter will then drop out by itself or you can give it a little pull to help it come out."





SITUATION

"Put the catheter back in the pack and throw it away. It should only be used once."



















































