SenSura[®] Mio 2-piece Click

Pictorial instructions for use

SenSura[®] Mio

 **Coloplast**


Abbreviated instructions for use. Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

Prepare — 1

Measure the stoma.

Prepare — 3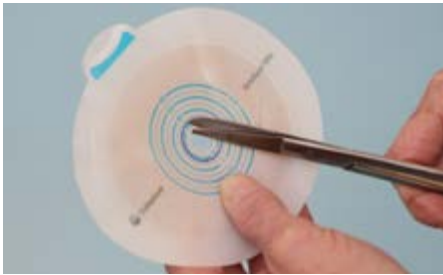

Cut opening in the barrier.

Apply — 1

Center barrier.

Apply — 3


Prepare — 2

Trace measurement onto back of the barrier.

Prepare — 4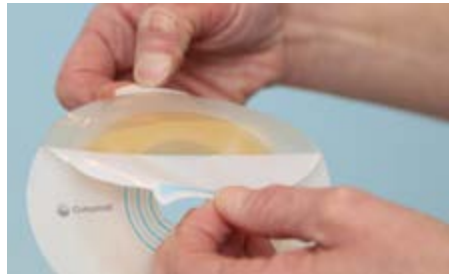

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply — 2

Place pouch on the barrier starting at the bottom flange. Gently press pouch into place while moving toward the top of the barrier with both hands.

Secure pouch by pinching the two ends of tab together on the click lock security ring. You will hear a click when it locks.

When pouch is $\frac{1}{3}$ to $\frac{1}{2}$ full, proceed to either **empty** or **remove**.

For drainable pouches only

Empty — 1

Open the outlet by lifting both tabs off the Velcro® plate. Unfold outlet.

Empty — 2

Fold back lower plate to avoid soiling when emptying. Attach Velcro® dots to hold plate in place.

Empty — 3

Empty pouch by pinching outer edges of the outlet open.

Empty — 4

Clean the outlet.

Close — 1

Release the Velcro® dots. Using the guide, fold the lower plate over the upper plate.

Close — 2

Fold upward. Seal the outlet by attaching the Velcro® tabs onto the Velcro® plate.

Remove - 1

Open the security ring by pressing the underside of the turquoise tab with your thumb or fingertip.

Remove - 2

Remove pouch from barrier by grasping the ring at the top of the pouch and pulling downward with a smooth "up-and-over" motion.

Remove - 3

Use the white tab to gently pull the barrier away from the skin using the "push-pull" technique.

For product support, call the Coloplast Consumer Care Team at 1-877-858-2656.

Coloplast develops products and services that make life easier for people with very personal and private medical conditions. Working closely with the people who use our products, we create solutions that are sensitive to their special needs. We call this intimate healthcare. Our business includes ostomy care, urology and continence care and wound and skin care. We operate globally and employ more than 10,000 people.

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2016 Coloplast Corp. All rights reserved.

Velcro® is a registered trademark of Velcro Industries B.V.

Features

Hide-away

Hide-away: Tuck sealed outlet up and under the protective pocket.

For extra discretion: Adhere the closed outlet on a drainable pouch to the Velcro® dot near inspection window.

Apply teal sticker to exposed part of the filter, on the back of the pouch, before showering, swimming or submerging in water. Remove sticker when done. Stickers come in the box of pouches.

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Velcro® es una marca comercial registrada de Velcro Industries B.V.