

Coloplast ostomy product decision guide

Finding the right ostomy solution starts with understanding the patient's ostomy type, body profile and stoma position. Use the information below to help guide your ostomy product selection.

Fecal stomas (colostomy and ileostomy)

1. What is the form of the area around the stoma? Choose: Regular, Inward, or Outward

R Regular
The area is essentially level with the abdomen.

I Inward
The area sinks into the abdomen, creating a hollow.

O Outward
The area rises from the abdomen, creating a peak.

2. What is the position of the stoma opening in relation to the surface of the skin? Choose: Above (protruding), Level (flush), or Below (retracted)

Above	Level	Below
-------	-------	-------

Above	Level	Below
-------	-------	-------

Above	Level	Below
-------	-------	-------

Suggested product

Flat or Soft Convex	Soft Convex or Convex Light	Convex Light or Deep Convex
---------------------	-----------------------------	-----------------------------

Soft Convex or Convex Light	Convex Light or Deep Convex	Convex Light or Deep Convex
-----------------------------	-----------------------------	-----------------------------

Convex Flip	Convex Flip and Brava® Protective Seal or Convex Light	Soft Convex, Convex Light or Deep Convex*
-------------	--	---

Urinary stomas (urostomy)

1. What is the form of the area around the stoma? Choose: Regular, Inward, or Outward

R Regular
The area is essentially level with the abdomen.

I Inward
The area sinks into the abdomen, creating a hollow.

O Outward
The area rises from the abdomen, creating a peak.

2. What is the position of the stoma opening in relation to the surface of the skin? Choose: Above (protruding), Level (flush), or Below (retracted)

Above	Level	Below
-------	-------	-------

Above	Level	Below
-------	-------	-------

Above	Level	Below
-------	-------	-------

Suggested product

Flat or Soft Convex	Convex Light or Deep Convex	Soft Convex or Convex Light
---------------------	-----------------------------	-----------------------------

Soft Convex or Convex Light	Convex Light or Deep Convex	Convex Light or Deep Convex
-----------------------------	-----------------------------	-----------------------------

Convex Flip	Convex Flip and Brava® Protective Seal or Convex Light	Soft Convex, Convex Light or Deep Convex*
-------------	--	---

*Note: Deep Convex is not recommended for use on firm, outward areas. Always consult a WOC Nurse if in doubt / before selecting Deep Convex.

Questions? Call your Coloplast Care Advisor at **1-877-858-2656**

Have questions or need product assistance? *We're here to help!*

Live support*: Call your Coloplast Care Advisor at **1-877-858-2656**

*7am – 7pm (CST), Monday-Friday



Guide your patients to a better life with an ostomy with Coloplast® Care

With the Coloplast Care program, patients receive:



Coloplast Care Advisor

Individualized *support* and *product access coordination* from a dedicated advisor



The Ostomy Check

Routine *self-assessments* to identify and address concerns



Educational resources

Personalized, *nurse-validated education*

- Coloplast Care Kit
- Website and emails

Enroll your patients at coloplastcareenroll.com

Ostomy Care / Continence Care / Wound & Skin Care / Interventional Urology

Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464

www.coloplast.us The Coloplast logo is a registered trademark of Coloplast A/S. © 2019 Coloplast Corp. All rights reserved.



PM-08891 10.19