SenSura[®] Mio 1-piece pouch

Pictorial instructions for use

SenSura[®] Mio

 **Coloplast**

Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

Prepare — 1

Measure the stoma.

Prepare — 2

Trace measurement onto back of the barrier.

Prepare — 3

Cut opening in the barrier.

Prepare — 4

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply — 1

Apply pouch by centering barrier around stoma.

Apply — 2

Secure to skin by applying gentle pressure. When pouch is $\frac{1}{3}$ to $\frac{1}{2}$ full, proceed to either empty or remove.

For drainable pouches only

Empty — 1

Open the outlet by lifting both tabs off the Velcro® plate. Unfold outlet.

Empty — 2

Fold back lower plate to avoid soiling when emptying. Attach Velcro® dots to hold plate in place.

Empty — 3

Empty pouch by pinching outer edges of the outlet open.

Empty — 4

Clean the outlet.

Close — 1

Release the Velcro® dots. Using the guide, fold the lower plate over the upper plate.

Close — 2

Fold upward. Seal the outlet by attaching the Velcro® tabs onto the Velcro® plate.

Remove — 1

To remove, use the white tab on the barrier to gently pull the pouch away from the skin.

Apply teal sticker to exposed part of the filter, on the back of the pouch, before showering, swimming or submerging in water. Remove sticker when done. Stickers come in the box of pouches.

Features

Hide-away

Hide-away: Tuck sealed outlet up and under the protective pocket.

For extra discretion: Adhere the closed outlet on a drainable pouch to the Velcro® dot near inspection window.

Inspection window

Lift the pouch fabric to inspect your stoma for easier placement upon application.

For product support, call the Coloplast Consumer Care Team at 1-877-858-2656.

Coloplast develops products and services that make life easier for people with very personal and private medical conditions. Working closely with the people who use our products, we create solutions that are sensitive to their special needs. We call this intimate healthcare. Our business includes ostomy care, urology and continence care and wound and skin care. We operate globally and employ more than 12,000 people.

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2019 Coloplast Corp. All rights reserved.

Velcro® is a registered trademark of Velcro Industries B.V.

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El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S.
© 2019 Coloplast Corp. Todos los derechos reservados.
Velcro® es una marca comercial registrada de Velcro Industries B.V.