SenSura[®] Mio 1-piece urostomy pouch

Pictorial guide

Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

Prepare — 1

Measure the stoma.

Prepare — 2

Trace measurement onto back of the barrier.

Prepare — 3

Cut opening in the barrier.

Prepare — 4

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply — 1

Before applying the pouch, check to ensure the drainable outlet has been securely fastened (plug to close). Then, apply pouch by centering barrier opening around stoma.

Apply — 2

Secure pouch by applying gentle pressure around the barrier.
***Note:** Per the WOCN® guideline, empty or remove the pouch when it's 1/3 to 1/2 full.

Empty — 1

Empty the urostomy pouch by holding the end of the outlet upwards. Pinch spout to control drainage and remove the plug. Direct the spout and release to drain.

Empty — 2

The Coloplast urostomy night bag can be attached for bedside drainage. If using a different drain bag, you may need an adaptor.

Remove — 1

To remove, use the white tab on the barrier to gently pull the pouch away from the skin.

For product support, call the Coloplast Consumer Care Team at 1-888-726-7872.

Coloplast develops products and services that make life easier for people with very personal and private medical conditions. Working closely with the people who use our products, we create solutions that are sensitive to their special needs. We call this intimate healthcare. Our business includes ostomy care, urology and continence care and wound and skin care. We operate globally and employ more than 12,000 people.

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2019 Coloplast Corp. All rights reserved.

Brava®

*I've found the solution
that is right for me.*

Louise*, Ileostomy since 2016

Brava® Protective Seal is right for Louise

What's right for you?

Brava® Protective Seal is one of the range of **Brava®** supporting products that can be used for customizing appliance fit and for issues such as leakage and skin irritation.

Looking for security and confidence again?

For more information, call Coloplast at
1-888-726-7872 or visit **www.coloplast.us**

*Louise is a Brava user who has received compensation from Coloplast to provide this information.

Stomy Care / Continence Care / Wound & Skin Care / Interventional Urology

Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464
www.coloplast.us The Coloplast logo is a registered trademark of Coloplast A/S. © 2019 Coloplast Corp. All rights reserved.

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El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S.
© 2019 Coloplast Corp. Todos los derechos reservados.