SenSura[®] Mio 2-piece Click Urostomy

Pictorial guide


Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

Prepare — 1

Measure the stoma.

Prepare — 3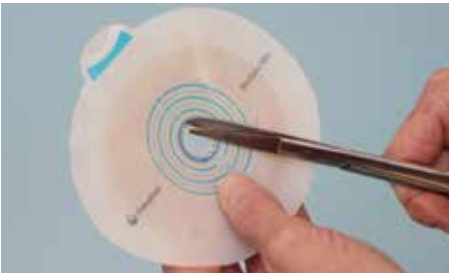

Cut opening in the barrier.

Apply — 1

Center barrier around stoma. Secure to skin by applying gentle pressure.

Apply — 3

Prepare — 2

Trace measurement onto back of the barrier.

Prepare — 4

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply — 2

Place pouch on the barrier starting at the bottom flange. Gently press pouch into place while moving toward the top of the barrier with both hands.

Secure pouch by pinching the two ends of the tab together on the click lock security ring. You will hear a click when it locks.

When pouch is 1/3 to 1/2 full, proceed to empty.

Empty — 1

Empty the urostomy pouch by holding the end of the outlet upwards. Pinch spout to control drainage and remove the plug. Direct the spout and release to drain.

Remove - 1

Open the security ring by pressing the underside of the turquoise tab with your thumb or fingertip.

Remove - 3

Use the white "soft-feel" tab on the barrier to gently pull the barrier away from the skin using the "push-pull" technique.

Empty — 2

The Coloplast urostomy night bag can be attached for bedside drainage. If using a different drain bag, you may need an adaptor.

Remove - 2

Remove pouch from barrier by grasping the top of the pouch and pulling downward with a smooth "up-and-over" motion.

**For product support, call the
Coloplast Consumer Care Team at
1-877-858-2656.**

Coloplast develops products and services that make life easier for people with very personal and private medical conditions. Working closely with the people who use our products, we create solutions that are sensitive to their special needs. We call this intimate healthcare. Our business includes ostomy care, urology and continence care and wound and skin care. We operate globally and employ more than 11,000 people.

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2018 Coloplast Corp. All rights reserved.
Velcro® is a registered trademark of Velcro Industries B.V.

Brava®

“Brava® Elastic Barrier Strips provide that *extra security.*”

Keagan*, 27

* Keagan is a Brava user who has received compensation from Coloplast to provide this information

Try the Brava® Elastic Barrier Strip XL

Keagan uses **Brava Elastic Barrier Strips** for a greater sense of security

Skin creases, folds and moisture can make keeping your ostomy barrier in place difficult. Skin-friendly Brava Elastic Barrier Strips keep barriers secure and prevent edges from lifting and rolling.

Now, a larger size is available!

Brava Elastic Barrier Strip XL provides an even larger adhesive area for a greater sense of security.

Brava® Elastic Barrier Strip XL

- ✔ Provides a larger adhesive area, for an increased feeling of security
- ✔ Supports longer wear time
- ✔ Absorbs moisture during movement

To learn more and request free samples

Call us toll-free at 1-833-806-9893 or go to www.coloplast.us/ebs-xl-offer

Ostomy Care / Continence Care / Wound & Skin Care / Interventional Urology

Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464

www.coloplast.us The Coloplast logo is a registered trademark of Coloplast A/S.
© 2018 Coloplast Corp. All rights reserved.

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El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S.
© 2018 Coloplast Corp. Todos los derechos reservados.
Velcro® es una marca comercial registrada de Velcro Industries B.V.
www.coloplast.us El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S.