# *SenSura<sup>®</sup> Mio 2-piece Flex*

Pictorial guide

*SenSura<sup>®</sup> Mio*

 Coloplast

Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

### Prepare — 1

Measure the stoma.

### Prepare — 2

Trace measurement onto back of the barrier.

### Prepare — 3

Cut opening in the barrier.

Remove the protective backing by pulling the turquoise release tab away from the barrier.

### Apply — 1

Center barrier around stoma.  
Secure to skin by applying gentle pressure.

### Apply — 2

Remove the protective backing from the adhesive ring on the pouch.

***For product support, call the Coloplast Consumer Care Team at 1-877-858-2656.***

### Apply — 3

Place pouch by aligning the bottom edge of the adhesive ring with the turquoise line at the bottom of the floating barrier.

### Apply — 4

Secure pouch by applying gentle pressure around the barrier.

\*Note: Per the Wound, Ostomy & Continence Nurse Society guideline, empty or remove the pouch when it's 1/3 to 1/2 full.

## For drainable pouches only

### Empty — 1

Open the outlet by lifting both tabs off the VELCRO® Brand fastener. Unfold outlet.

### Empty — 2

Fold back lower plate to avoid soiling when emptying. Attach VELCRO® Brand fastener dots to hold plate in place.

### Empty — 3

Empty pouch by pinching outer edges of the outlet open.

### Empty — 4

Clean the outlet.

## Close — 1

Release the VELCRO® Brand fastener dots. Using the guide, fold the lower plate over the upper plate.

## Close — 2

Fold upward. Seal the outlet by attaching the VELCRO® Brand fastener tabs onto the fastener strip.

## Remove - 1

Remove pouch by using the turquoise tab located on the adhesive ring and gently pull down and away.

## Remove - 2

Use the white tab to gently pull the barrier away from the skin using the "push-pull" technique.

## Features

### Hide-away

**Hide-away:** Tuck sealed outlet up and under the protective pocket.

**For extra discretion:** Adhere the closed outlet on a drainable pouch to the VELCRO® dot near inspection window.

Apply teal sticker to exposed part of the filter, on the back of the pouch, before showering, swimming or submerging in water. Remove sticker when done. Stickers come in the box of pouches.

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