SenSura® Mio

 **Coloplast**

SenSura® Mio 2-piece Flex Uro

Pictorial guide


Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

Prepare — 1

Measure the stoma.

Prepare — 2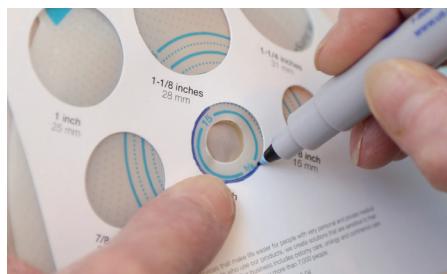

Trace measurement onto back of the barrier.

Prepare — 3

Cut opening in the barrier.

Prepare — 4

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply — 1

Center barrier around stoma. Secure to skin by applying gentle pressure.

Apply — 2

Remove the protective backing from the adhesive ring on the pouch.

Apply — 3

Place pouch by aligning the bottom edge of the adhesive ring with the turquoise line at the bottom of the floating barrier.

Apply — 4

Secure pouch by applying gentle pressure around the barrier.

*Note: Per the Wound, Ostomy & Continence Nurse Society guideline, empty or remove the pouch when it's 1/3 to 1/2 full.

Empty — 1

Empty the urostomy pouch by holding the end of the outlet upwards. Pinch spout to control drainage and remove the plug. Direct the spout and release to drain.


Empty — 2

The Coloplast urostomy night bag can be attached for bedside drainage. If using a different drain bag, you may need an adaptor.

**For product support, call the Coloplast
Consumer Care Team at 1-877-858-2656.**

Remove - 1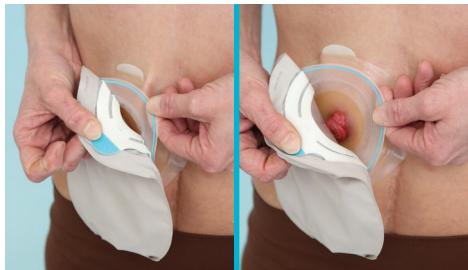

Remove pouch by using the turquoise tab located on the adhesive ring and gently pull down and away.

Remove - 2

Use the white tab to gently pull the barrier away from the skin using the "push-pull" technique.

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www.coloplast.us El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S. © 2019 Coloplast Corp. Todos los derechos reservados.
Coloplast Corp. Minneapolis, MN 55411 | 1-800-533-0464
Velcro® es una marca comercial registrada de Velcro Industries B.V.